



Aerosol Capture Kit Assembly Instructions

Features of the Aerosol Capture Kit



Kit Assembly

1. Remove the tent components from the plastic bag
2. Unfold the sides of the tent
3. Snap the separate panel to the main tent to complete construction.
4. If desired, place the end of the 2' of corrugated tubing through the top of the tent.
5. Attach the T-piece connector to the end of the tubing to disperse the air flow.
6. Place the patient's torso within the tent on a pillow for comfort.
7. Remove the miniSQUAIR® from its pouch, remove the release paper and adhere to the pillow. Lead the tubing from the tent beneath the flap.
8. Fold down the front flap to complete patient isolation.
9. Connect corrugated tubing to air mixture and miniSQUAIR® tubing to ULPA filter of the suction device.

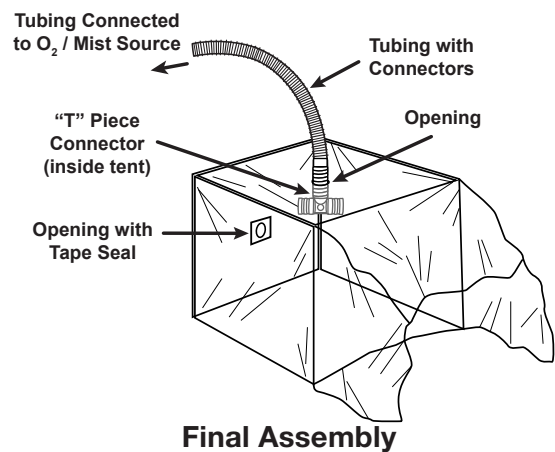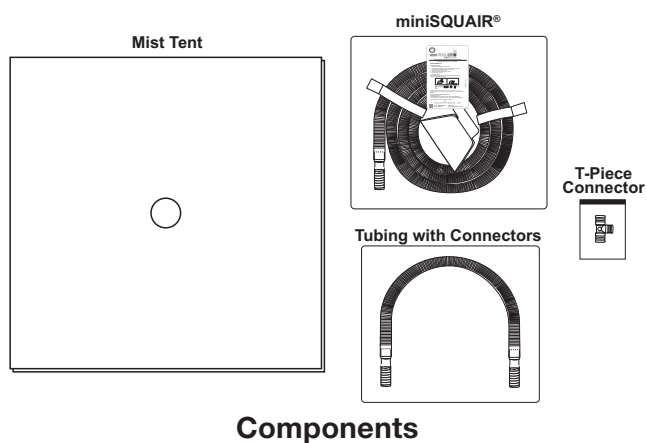

Contact: 952-345-1112, info@nascentsurgical.com, NascentSurgical.com