



Isolation Tent with Access Ports Assembly Instructions

Features of the Isolation Tent with Access Ports

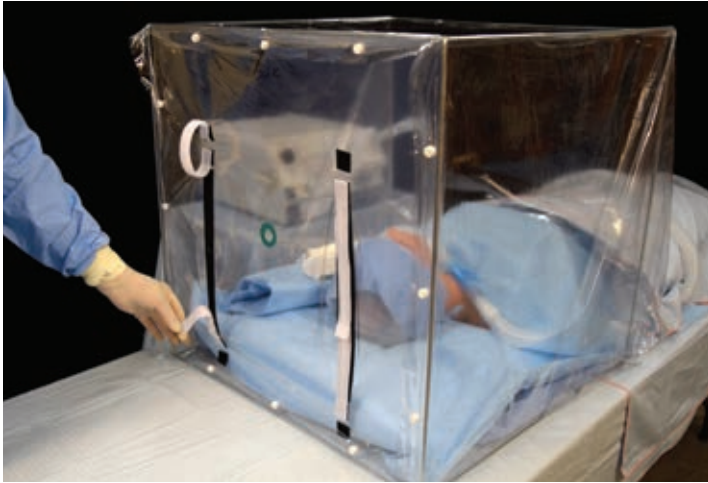


Velcro Tabs are Rolled Up from Access Port Openings

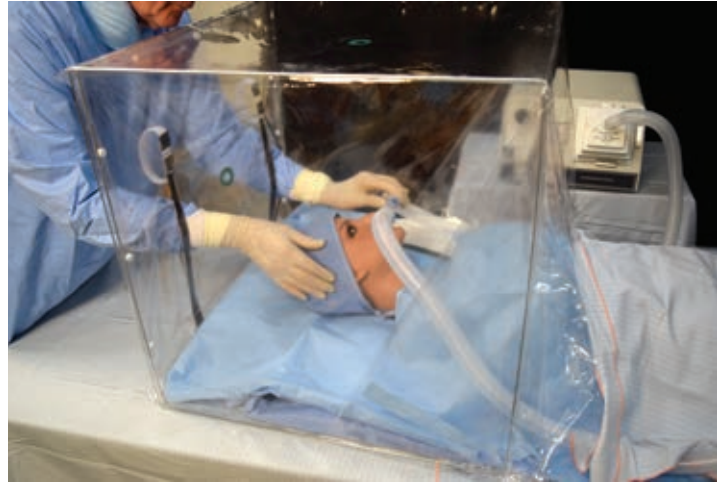


Port Openings Allow Access to Patient

Tent Assembly

1. Remove the tent components from the plastic bag
2. Unfold the sides of the tent
3. Snap the panel containing the re-sealable access ports to the main tent to complete construction.
4. Place the patient's torso within the tent.
5. Fold down the front flap to complete patient isolation.
6. Roll up the white tabs that cover the ports and attach them to the tabs at the ends of the ports.
7. Proceed with intubation through the access ports.
8. Reseal the ports once the endotracheal tube is taped in place and connected to respiratory tubing.

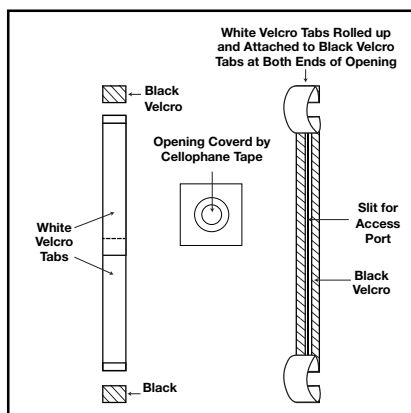

Back Panel with Access Ports

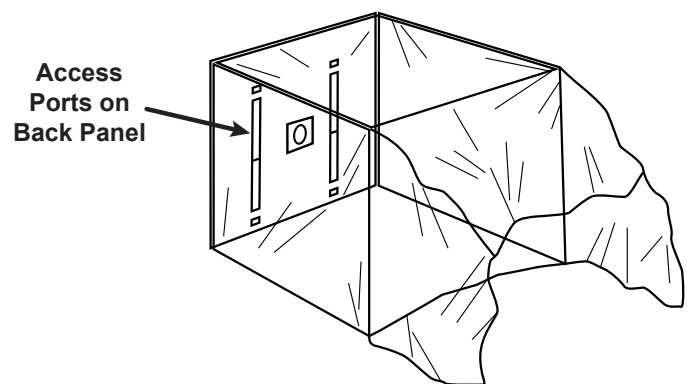

Final Assembly

Contact: 952-345-1112, info@nascentsurgical.com, NascentSurgical.com